



Health And Hydrogen

# Operating Manual

## HAH-1800



# Usage Precautions

## Instructions for First Use and Adding Water

Before the initial use of the product, please add distilled water to the machine's water tank and let it sit for 5 minutes. Then, open the drainage outlet, drain for approximately 10 seconds, close the drainage outlet, and start the machine.

When adding water to the machine, please aim at the water tank opening and avoid overfilling to prevent water from overflowing into the interior of the machine, which could lead to equipment damage.



Notice

**Recommendation:** Use disposable tubes, assign them for individual use, and if you intend to reuse a tube multiple times, please sterilize it in hot boiling water and then reuse it, or replace it with disposable nasal accessories.



Notice

## There is Water Dew in the Machine

Regarding the machine exhibiting water dew: Each machine undergoes testing before leaving the factory, and there might be a little water in the window or in the water tank. If you receive a machine with water dew, this is normal. If you notice excessive water dew on the machine, simply wipe it off with a dry cloth and allow it to dry.



Notice

Before using it daily, please empty the water from the water container to prevent an excessive amount of water in the water trap from flowing into the nostril pipe and affecting the gasflow.



Notice

## Change the water

Suggestion: When the machine is not used for an extended period or is used frequently, it is recommended to change the water once a week.



Notice

## Ensure Timely Heat Dissipation

The machine should not operate continuously for more than 8 hours. Please turn off the power switch after 8 hours of operation, allow the machine to dissipate heat, and restart it after a 1-hour break. It is not advisable to run it continuously for extended periods to prevent damage to the machine.



Notice



## WARNING

1. Do not insert or manipulate plugs with wet hands.
2. Do not use water, liquids, cleaners, or any flammable substances to clean or spray on this product.
3. Do not use an extension cord unless approved by local authorities.
4. Do not attempt to disassemble, repair, or modify the equipment.
5. If not using this product for an extended period, please unplug the power plug.
6. Do not use this product in environments with significant temperature fluctuations.
7. If this product emits abnormal sounds, burning odors, or smoke, immediately disconnect the power plug in a safe manner and contact the technical support center.
8. If the plug is not suitable for the socket, please contact qualified personnel to install an appropriate power socket.
7. Do not arbitrarily change plugs or use adapters.

# Safety Notice

## Keep away from fire

Smoking is prohibited during use, and open flames are strictly prohibited to prevent any potential danger



Prohibition

## It is forbidden to repair or disassemble the product without authorization.

Non-professional maintenance personnel are prohibited from privately repairing and disassembling the machine, as it may result in fire, electric shock, and other hazards.



Prohibition

## Confirm voltage

Before use, please ensure that the voltage meets the product's requirements.



Notice

Please do not place any sundries around the machine to prevent blocking the gas outlet, which can lead to excessive temperature and cause the machine to shut down or a decrease in the concentration of gas. Grease or grease-like substances must not be used in any part of the machine to avoid contaminating the gas outlet and posing a danger. When the machine is in operation, all gas outlets must be kept unobstructed. Do not block the gas outlet or bend the nasal pipe.



Prohibition

## Ventilation

The machine should be positioned indoors in a well-ventilated area, away from direct sunlight.



Prohibition

## The product must not be turned upside down or laid flat.

During transportation, it is strictly prohibited to place the machine upside down. When it is not in use, please promptly drain and replace the water. If you need to transport it, ensure that the water tank is completely emptied to prevent damage to the machine.



Prohibition

## Prohibit to use non-original accessories

Using non-original accessories for the machine is prohibited to prevent damage to the machine.



Prohibition

Please select an appropriate indoor environment when the machine is in operation. The room temperature should not be lower than 10°C or higher than 30°C, as excessively low or high temperatures may result in the machine not functioning properly.



Notice

When the machine is not in use for an extended period, please turn off the power and disconnect the power plug.



Notice



## Safety Precautions

1. Ensure that the power plug is connected to a properly labeled power outlet with the voltage matching the rating plate.
  2. If the wire becomes damaged, it must be replaced by a qualified service technician.
  3. Verify that the power outlet is correctly connected, and avoid using the plug under heavy loads to prevent overheating.
  4. Maintain a minimum distance of 20-30 cm between the machine and walls or other objects.
  5. Avoid pushing, bending, squeezing, hitting, or inserting sharp objects into any part of the machine.
  6. Turn off the power before cleaning or performing maintenance on the machine.
  7. Please unplug the power before cleaning the machine.
  8. To clean the product's body, use a soft, dry cloth to remove dust and dirt from the surface.
- Tip: If the contamination is severe, you can use a damp cloth to gently wipe off the surface, but do not use rough materials.



# Advantages



One Click Run



Beauty accessories



Flow rate Adjustment



1-8 hours timing



Hydrogen only



oxygen only



oxy-hydrogen





hydrogen water

## PRODUC SPECIFICATIONS

<b>Product Type</b>	<b>99.99% Hydrogen</b>	<b>Oxy-hydrogen 2:1</b>
<b>Flow Rate</b>	<b>300/600 ml/min</b>	<b>450/900/1500/1800ml/min</b>
<b>Size</b>	<b>193*285*300mm</b>	
<b>Noise</b>	<b>&lt;50db</b>	
<b>Voltage</b>	<b>110V/220v</b>	
<b>Capacity</b>	<b>&lt;150-350W</b>	

# Product Introduce

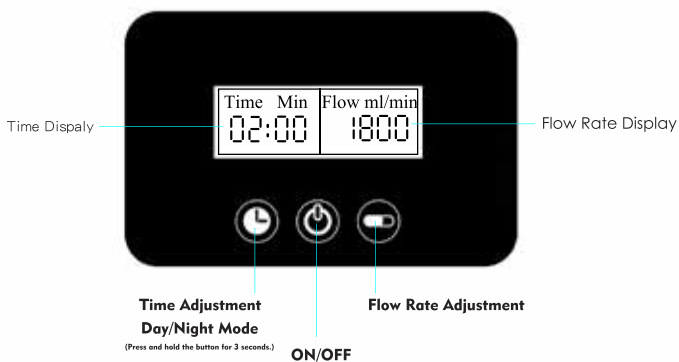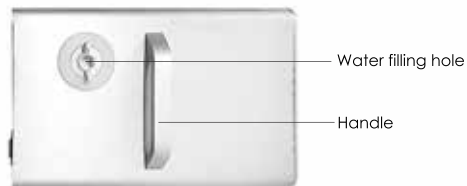

\*The diagrams in this manual are for reference only, please refer to the actual product

\*If there is any technical change to this product in the future, such as the appearance and color of the product, it is subject to change without notice.

\*This product is limited to normal operation and must be used in accordance with the instruction manual

# Quick Operation Guide



Step 1. Connect the power supply, turn on the power switch, the red light is always on.



Step 2. Hold the handle and turn it counterclockwise to unscrew the tank cover as shown.



Step 3. After unscrewing the water tank cover, add the pure water with the level of 5-6cm below the mouth of the water tank. The device gives the voice prompt: The water level is high



## Step.4 Important :

When using the device for the first time,  
1) fill the tank with water and keep the tank cover open.  
2) Turn the drain valve at the back of the machine to drain water for 10-15 seconds. (When draining water, pay attention to observe the state of the water, water must be released until the water flow continues to flow and without bubbles.)

During regular use, simply add pure water to the machine on schedule.



Step 5. Click the on-off button to running the machine.



Step 6. Lift-Click as instructions to adjust usage time or day/night mode.  
Right-click to adjust machine flow rate.




Step 7. Connect the water container accessories as shown in the diagram.



Step 8. During your use, if any water remains in the water collector, it can be rotated and separated to two sides according to the "Figure" to pour out the excess water.


# Screen Display Instruction

---

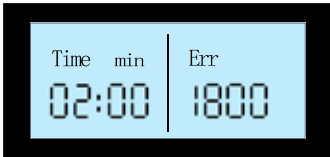


Err

Voice Prompt: best water level is reached, please stop adding water

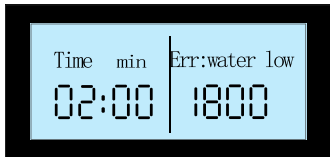

Err:water low

Voice Prompt: the water level is low, please add water in time

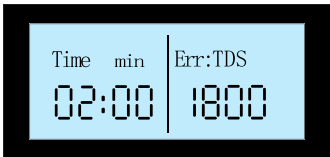

Err:TDS

Voice Prompt:the water quality is not qualified, please replace pure water



Err:overheat

Voice Prompt:the temperature is high, please stop using



Err:pressure

Voice Prompt: the hydrogen pressure is high, please check whether the hydrogen suction pipe is bent or squized



Dear customers,

Thank you for purchasing our products. Please ensure to read the instructions carefully before use the machine so as to avoid unnecessary damages. In case of failure, you can consult the after-sales service department of our company, and we will provide you with excellent maintenance service.

We will give you a reply in the shortest possible time. Thank you very much!

### **After-sales service instructions**

If the machine suffers a troubleshooting process

1. Please contact the after-sales service and consult the relevant staff.
2. The product purchased for in 1 year can be guided or returned for repair with free maintenance, and the purchaser shall pay the shipping cost.
3. The product purchased for more than 1 year can be guided or returned for repair, and the purchaser shall pay the maintenance cost, shipping cost, etc.

# **Warranty Card**

**Product Model**

**Purchasing Date**

**Customer Name**

**Address**

**Maintenance Record**



Health And Hydrogen

**Address:**

Unit 908, Prosperity Millenia Plaza,  
663 King's Road, Quarry Bay,  
Hong Kong Island, Hong Kong